

## LIGHTGUARD SIZING GUIDE INSTRUCTIONS


This simple to use sizing guide will help you select the LightGuard OveRx size that most accurately fits over your prescription eyewear. To use the guide, set your print preferences with no scaling and print the document at **100% actual size**. Then place your eyewear face down on the images of each size. If your eyewear fits within the blue box, the size can be worn over your glasses. If your eyewear exceeds the box, then you must try the next size up. The ideal fit will come from the size that is just slightly larger than your prescription eyewear.



The image of the quarter should be the same size as an actual quarter when printed at full scale. The image should be 24 mm in diameter at full size.

### LIGHTGUARD Medium (M)

Fits over frames up to 5 3/8" w x 1 3/4" h (136 mm x 44 mm)

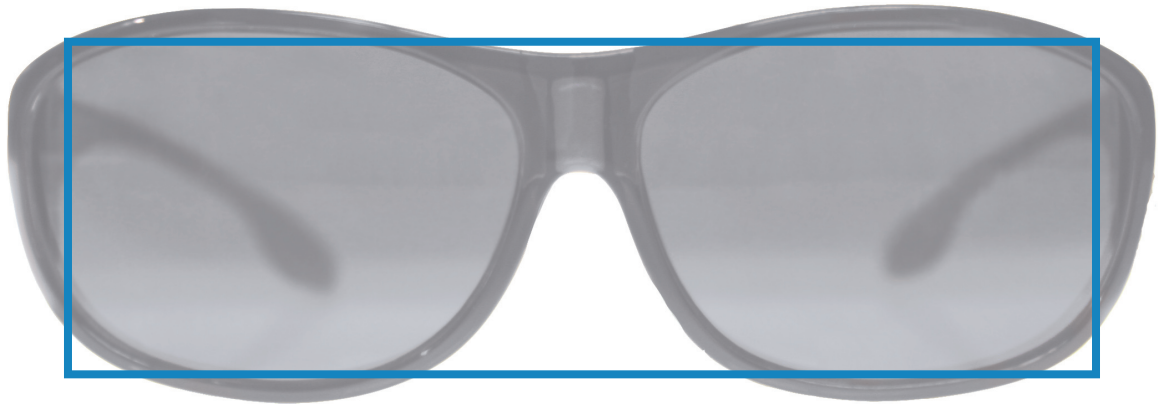

### LIGHTGUARD Large (L)

Fits over frames up to 5 1/2" w x 2" h (140 mm x 50 mm)

